

VAXOL®

Ear spray

Vaxol® is an easy and effective method for naturally softening and removing ear wax. It reduces the need for ear syringing and can make ear syringing easier when it is required. It has been clinically proven that olive oil moisturises dry skin, and so reduces itching and irritation.

Vaxol® can be used by adults and children aged over 1 year.

Vaxol has a simple dosing system that makes it easy to deliver the correct dose into the outer ear.

Vaxol® can be used sitting or standing.

Read this package leaflet before use.

BEFORE YOU USE A NEW BOTTLE OF VAXOL® SPRAY

Before using Vaxol® spray, warm the bottle in your hand and shake it well. Spray 3-4 times into the air until you get a steady spray (see illustration B). Vaxol® is now ready to be used.

INSTRUCTIONS

- Vaxol® is most effective when it has been warmed in the hand and shaken well before use.
- Use 1-2 times a week to prevent wax plugs or use Vaxol® according to the doctor's / nurse's instructions.
- If medical ear syringing is necessary (wax plugs), spray 1-2 doses of Vaxol twice daily for 4-5 days, or as directed , before seeing a doctor/nurse.
- For itching and irritation in dry auditory canals, spray once or twice a week or as directed.
- Carefully pull the top of the outer ear up and back (see illustration A).
- Hold the bottle upright with the spray nozzle in the outer auditory canal and press the spray head fully down once or twice (see illustration B).
- Carefully massage the opening of the outer auditory canal (tragus, see illustration A).
- Repeat the procedure in the other ear, if required.
- Do not rinse the ear after spraying.
- If required, the spray nozzle can be removed and rinsed in warm water after use. Afterwards, replace the nozzle on the bottle.

FURTHER INFORMATION

- Patients with a perforated eardrum or earache should only use Vaxol® following contact with a doctor.
- Vaxol® must not be used in children aged less than one year without contact with a doctor.
- Olive oil is a natural product with a wide range of uses. There are no reports of side effects from Vaxol®, including when used together with other products. Vaxol® must not be used if you are hypersensitive to olive oil.
- There are no known harmful side effects from using Vaxol® during pregnancy or when breastfeeding. However, you should inform your doctor that you are using Vaxol®.
- There are no known side effects from overdose of Vaxol®.
- In case of a suspected allergic reaction, stop the treatment with Vaxol® and contact a doctor immediately.

WARNINGS

- Vaxol® is intended for personal use only.
- Always store Vaxol® upright between 15 – 25 Deg C.
- Vaxol® can be stored for 12 months after the spray bottle has been opened.
- Do not spray near to naked flames.

V119-16

- Keep out of the sight and reach of children.
- Vaxol® is only intended for use in the outer auditory canal. Do not spray near to the eyes, nose or mouth.

Consumer contact:

Farmacia Nordic AB, www.farmacianordic.com

The package leaflet was revised in 03/2024

Vaxol is a medical technical product class I which conforms to EU Directive 93/42/EEC.

www.vaxol.com

Manufactured by: HL Healthcare Ltd, Pure Offices, Plato Close Tachbrook Business Park,
Warwick, CV34 6WE, United Kingdom



Advena Ltd. Tower Business Centre, 2nd Fl.,
Tower Street, Swatar, BKR 4013 Malta

Imported and Distributed by: **Farmacia Nordic AB**
Ekonomivägen 5, 436 33 ASKIM

V119-16 LAB001645